



**Practitioners Have Asked For a Multifocal GP Lens
That Will Minimize Corneal Moulding.
Here Is The Answer!**

The LifeStyle UltraNear Multifocal
A Front-Surface Multifocal Lens with a Spherical Base Curve

The fitting technique for The LifeStyle **UltraNear** Multifocal is the same as fitting a spherical lens. This lens should be fit on or near the Flat K Reading.

A well-fit lens will have centration over the pupil with some movement with the blink. The fluorescein pattern will show pooling in the center, giving the lens a mid-peripheral alignment.

A lens which has been fit too flat will have minimal apical clearance with nasal/temporal decentration. A lens which has been fit too steep will have apical vaulting.

Base Curve Range

6.80 mm—8.90 mm

Diameters

9.0, 9.5, 10.0 and 10.5
9.5 is Standard

Distance Optical Zone

3.0 to 5.0 in 0.25mm Steps
3.5 is Standard

Add Powers

+1.00 to +3.00
in +0.25D Steps

The LifeStyle Gp Company, LLC
7500 West Mississippi Avenue Suite E240
Lakewood, CO 80226

2530 Trailmate Drive
Sarasota, FL 34243

Phone: 888-379-6645
Local: 303-462-3793
Fax: 303-462-3836
Consultation Email:
smartsmarsource@aol.com